

IRRISEPT® PRODUCT INFORMATION


Irrisept is a jet lavage system, containing low concentration chlorhexidine gluconate (CHG*) 0.05% in sterile water for irrigation

Model # ISEPT-450-USA contents include:

- (1) Irrisept®, 450mL bottle, 0.05% CHG in sterile water, USP (99.95%)
- (1) Irriprobe® applicator

Contents are wrapped in two sequential CSR wraps and sealed in TYVEK® header pouch.


*FDA-Cleared (K080779) Class II Medical Device. *CHG acts as a preservative to help inhibit microbial growth in the solution.*

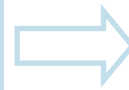


Remove TYVEK® strip to open dust bag/pouch. Recommend placing double wrapped CSR on flat surface to open. Irrisept bottle and Irriprobe applicator are loose together inside 2nd CSR wrap.



Grasp bottle top firmly and twist quickly in **ONLY ONE** direction, until seal cap snaps off.

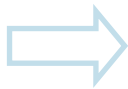

Attach applicator to bottle

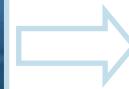

Wait approximately one minute, rinse with normal saline for irrigation.

Customer Service: 770-807-8445

cs@irrisept.com

www.irrisept.com

See reverse for important information

IRRISEPT® PRODUCT INFORMATION

INDICATIONS FOR USE

The mechanical action effectively loosens and removes wound debris.

WARNINGS

- Do not use this product if the patient is allergic to chlorhexidine gluconate.
- Discontinue use immediately if irritation, sensitization or allergic reaction occurs.

CAUTIONS

- Do not use unless solution is clear and bottle twist seal is intact.
- When using this product keep away from eyes and ear canals. If the solution should contact these areas, rinse out promptly and thoroughly with water and/or normal saline.
- Federal law restricts this device to sale by or on the order of a licensed physician.
- Not for injection.
- Single patient use only.

FEATURES

- Not made with natural rubber latex
- Single use, disposable
- Self-contained, portable



Customer Service: 770-807-8445

cs@irrisept.com

www.irrisept.com

Page 2 of 2
IRR120 REV 01